



1. Connect transducer(s) (cable point up).
2. Press **On** button (above-right of transducer connectors).
3. Press **New Patient**
4. Enter patient information and select **Exam/Pres** on this screen.
5. Press **Freeze** to begin imaging.
6. Press **Transducer** to display softkeys. Press softkey for desired transducer.
7. Press **Exam Type**, then select softkey for desired exam/preset.
8. Begin scanning and acquire desired image.
9. Adjust **Depth**.
10. Press **Optimize** to provide best image automatically.
11. Press **Freeze** to stop live scanning.
12. Use **Store/Print** to capture images/clips.

Review Images in Current Exam

1. Press **Current Exam** (QWERTY keyboard).
2. All stored images for exam appear.
3. To view an image in large format, point with cursor and press **Set** key.
4. Press **Current Exam** to exit review & return to live imaging.

Restart Closed Exam

1. Press **New Patient** (QWERTY keyboard).
2. Click **Restart** (bottom of screen).
3. Point cursor to desired exam and press **Set** key.

4. Click **Select Exam** (bottom of screen). Next, press **Exit** on patient form.
5. You can now resume exam & add images to the study.
6. Press **End Exam** (QWERTY keyboard) to end exam.

Delete Exams from Archive

NOTE: System has limited memory; delete/transfer exams weekly.

1. Press **Archive** (QWERTY keyboard).
2. Point cursor to desired exam & press **Set** key.
3. Click **Delete** (bottom of screen). Select **Yes** to confirm. Exam is deleted from hard drive.



Basic Measurements (Meas Button)

Depth (cm)

1. During *live* 2D scanning, press **Measure**.
2. Position caliper with **Trackball**.
3. Depth will display on screen.

Distance (cm)

1. Press **Measure** while 2D or M-Mode image is *frozen*.
2. Select **Distance** on menu & press **Set**.
3. Position first caliper with **Trackball** & press **Set**.
4. Position second caliper with **Trackball** & press **Set**.
5. Distance between calipers will display on screen.

Basic Calculations (Calc Button)

1. Press **Calc** while image is frozen.
2. Select desired calc on menu & press **Set**.
3. Position caliper with **Trackball** & press **Set**.
4. Position additional caliper(s) with **Trackball** & press **Set**. Calc will display on screen.
5. Press **Enter** or **Store** to store the result.
6. Press **Report** to display calcs you've recorded.

For use with z.one_{pro} Ultrasound Systems. For further information, see the z.one_{pro} Ultrasound Systems IFU (p/n Q00350-00).

Export Exams to USB or CD/DVD

1. Press **Archive** (QWERTY keyboard).
2. Point to **Export**. This will open Archive Exam Export.
3. Point to **Destination**. Then click **Export** at bottom of screen.
4. On next screen, click **Options**. **Archive Exam Export Options** screen displays:

5. Select **Non-DICOM**. Select **JPEG** or **TIFF**.
6. Click **Apply**.
7. Click **Export** on next screen. Verify export (**Yes/No**) at prompt.
8. If **Yes**, exam exports to USB/CD/DVD but is not deleted from hard drive.