



1. Connect Transducer(s) (cable pointing up)
2. Press On Button (above-right of transducer connectors)
3. Press New Patient
4. Enter patient information and select Exam/Pres on this screen
5. Press Freeze to begin imaging
6. Press Transducer to display Softkeys. Press softkey for desired transducer.
7. Press Exam Type, then select softkey for desired exam/preset.
8. Begin scanning and acquire desired image.
9. Adjust Depth.
10. Press Optimize to provide best image automatically
11. Press Freeze to stop live scanning
12. Use Store/Print to capture images/clips

Review Images in Current Exam

1. Press **Current Exam** (QWERTY keyboard).
2. All stored images for exam appear.
3. To view an image in large format, point with cursor and press **Set** key.
4. Press **Current Exam** to exit review & return to live imaging.

Restart Closed Exam

1. Press **New Patient** (QWERTY keyboard).
2. Click **Restart** (bottom of screen).
3. Point cursor to desired exam and press **Set** key.

4. Click **Select Exam** (bottom of screen). Next, press **Exit** on patient form.
5. You can now resume exam & add images to the study.
6. Press **End Exam** (QWERTY keyboard) to end exam.

Delete Exams from Archive

NOTE: System has limited memory; delete/transfer exams weekly.

1. Press **Archive** (QWERTY keyboard).
2. Point cursor to desired exam & press **Set** key.
3. Click **Delete** (bottom of screen). Select **Yes** to confirm. Exam is deleted from hard drive.



Basic Measurements (Meas Button)

Depth (cm)

1. During *live* 2D scanning, press **Measure**.
2. Position caliper with **Trackball**.
3. Depth will display on screen.

Distance (cm)

1. Press **Measure** while 2D or M-Mode image is *frozen*.
2. Select **Distance** on menu & press **Set**.
3. Position first caliper with **Trackball** & press **Set**.
4. Position second caliper with **Trackball** & press **Set**.
5. Distance between calipers will display on screen.

Basic Calculations (Calc Button)

1. Press **Calc** while image is frozen.
2. Select desired calc on menu & press **Set**.
3. Position caliper with **Trackball** & press **Set**.
4. Position additional caliper(s) with **Trackball** & press **Set**. Calc will display on screen.
5. Press **Enter** or **Store** to store the result.
6. Press **Report** to display calcs you've recorded.

Export Exams to USB or CD/DVD

1. Press **Archive** (QWERTY keyboard).
2. Point to **Export**. This will open Archive Exam Export.
3. Point to **Destination**. Then click **Export** at bottom of screen.
4. On next screen, click **Options**. **Archive Exam Export Options** screen displays:

5. Select **Non-DICOM**. Select **JPEG** or **TIFF**.
6. Click **Apply**.
7. Click **Export** on next screen. Verify export (**Yes/No**) at prompt.
8. If **Yes**, exam exports to USB/CD/DVD but is not deleted from hard drive.